



Accent® with Empower™: LAMP Words for Life® Quick Reference Guide

Empower Software Version 1.0 2019-03-01 or higher

PRC Regional Consultants: (800) 848-8008

PRC Tech Support: (800) 262-1990

service@prentrom.com

www.prentrom.com/support



Menu

Select **Menu** to gain access to the Quick Menu. The Quick Menu contains links to the Editor, Vocabulary Builder, and the Toolbox. It will also give you access to the Windows® Desktop if your Accent device is integrated.



POWER/SLEEP BUTTON

The power/sleep button is on the top of the device.

1. To turn on the device, press and hold power/sleep.
2. To wake/sleep the device, press power/sleep.
3. The device should be restarted weekly.*
4. To restart, select **Menu**, **Toolbox**, **System Settings**, **Power Settings**, **Restart**, and then **Yes**.

*** Recommended: Restart once per week; put to sleep daily. The device may wake up from sleep if using eye tracking or if Windows updates are set to install automatically. To preserve the charge overnight, either connect the charger or shut down the device completely. See step 5.**

5. If you don't plan to use the device, want to store/transport it in a bag, or receive an error message, shut down the device completely. To do this, select **Menu**, **Toolbox**, **System Settings**, **Power Settings**, **Shut Down Device**, and then **Yes**, shut down.

SELECT A VOCABULARY

1. Select **Menu**, and then **Toolbox**.
2. Select the **Vocabulary** tab, then **Select a Vocabulary**.
3. Select the **LAMP WFL** tile. Choose your desired LAMP WFL vocabulary. Note: Once you edit a button or create a Vocabulary Builder list, the vocabulary will be automatically saved and placed in **My_vocabularies**.

VOCABULARY SETTINGS

In this tile you can rename your vocabulary, change the font and font size on the buttons, change the background color and set button preferences.

1. Select **Menu**, and then **Toolbox**.
2. Select the **Vocabulary** tab, then **Vocabulary Settings**.

RENAME A VOCABULARY

From the **Settings** tab, edit the **Vocabulary Name**.

SET BUTTON PREFERENCES (recommended settings in **bold**)

From the **Button Preferences** tab, set the following:

Empty Button Appearance: What do you want empty buttons to look like? **White**, Transparent, Empty

Empty Button Action: What do you want empty buttons to do? **No Action**, Go To Home

Show Labels: Where do you want labels showing? **Home**, **Middle**, **End**

Use Alternative Icons: Where do you want an alternate set of icons to show? None, Middle, End, **All**

CHANGE THE VOICE

1. Select **Menu**, and then **Toolbox**.
2. From the **User Settings** tab, select **Voice**, then **Change Voice**.

EDIT A PRONUNCIATION

"Teach" the device to pronounce a word correctly.

1. Select **Menu**, and then **Toolbox**.
2. From the **User Settings** tab, select **Voice**.
3. Select **Edit a Pronunciation**.
4. Enter the correct spelling of the word, then **Finished**.

5. Enter the phonetic spelling of the word. To test the pronunciation, select the **Message Window**. When satisfied, select **Finished**, then **Add**.

WORD FINDER

Use Word Finder to locate a word in the vocabulary.

From the spelling keyboard:

1. Select **Word Finder**.
2. Type the word you want to find.
3. Select **Search**.
4. Select **Guide Me**. If the word is not in LAMP WFL, it will say "no results found".

EDIT A BUTTON

1. Select **Menu**, and then **Editor**.
2. Navigate to the button you want to edit, then select **Edit**. Select the button you want to edit.
3. Add or edit the **Message** and the **Label**.
4. Suggested symbols will appear. If you do not see one you like or you would like to take a **photo** select the **Symbol** tab. You can **Search**, **Browse**, or **Import** symbols or select **Camera**.
5. Additional button options:
 - a. **Colors:** Set various colors for the button.
 - b. **Actions:** Add an action. Action options include things such as Clear Display, Caps Lock, and Stop Speech.
 - c. **Settings:** Change button level settings. Settings include Message, Speech, Always Show Button Label, Alternate Pronunciations, and Alternate Prompt.

CREATE A BUTTON WITH A PAGE LINK


1. Select **Menu**, and then **Editor**.
2. Navigate to the button where you want a page link, then select **Edit**. Select the button you want to edit.
3. Add a **Label (in all upper case)** and a **Symbol**.
4. If desired, change the button color from the **Colors** tab.
5. Select the **Navigation** tab.
6. If the button already has navigation, select **Remove Navigation**. Select **Go To Page**.
7. Select **Add New**.
8. **Name your page** and **set the grid size (84 buttons)**.
9. Select **Create**.
10. Make sure **Make Persistent** is set to **OFF**.
11. Select **Finished**.
12. Navigate to your new page and add the words you want. See **Edit a Button**.

VOCABULARY BUILDER (VB)

Use Vocabulary Builder to temporarily limit the number of words appearing on the screen. The strategy of “masking” vocabulary allows a small amount of vocabulary to be taught on the device while keeping learned motor plans constant. For example, you want to start with 8 words when first introducing the device to a child.



1. Select **Menu**, and then **Vocabulary Builder**.

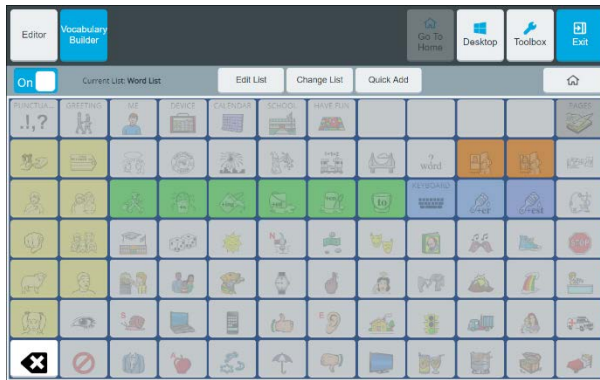

2. There are two ways to create a VB list of words.
 - a. Type a list of words.
 - i. Select **Edit List**.
 - ii. If desired, rename your list by using the text box next to **Name of Current List**.
 - iii. Select the text field and **type your word list separated by commas**.
 - iv. Select **Finished** and **Exit**.
 - b. Touch the icon sequences/symbols that lead to the word.
 - i. **Touch the pathway to the first word**. Select the home button (top right) when you need to return to the home screen.
 - ii. Continue to touch the pathways to words.
 - iii. Select **Exit** when finished.
3. Lists are automatically saved.

Additional VB Options:


Edit list: Create, rename, manage, and clear lists.

Quick Add: Use this to quickly add words to your list.

Change List: If you have multiple lists, you can switch between them. In addition, rename, duplicate, merge, and delete lists.

My Lists and Default Lists: In Default lists, find the LAMP WFL Transition list and activity-specific lists (such as cars, bubbles etc.).


Merge Lists: If you have multiple lists, you can combine as many as you want into one list.

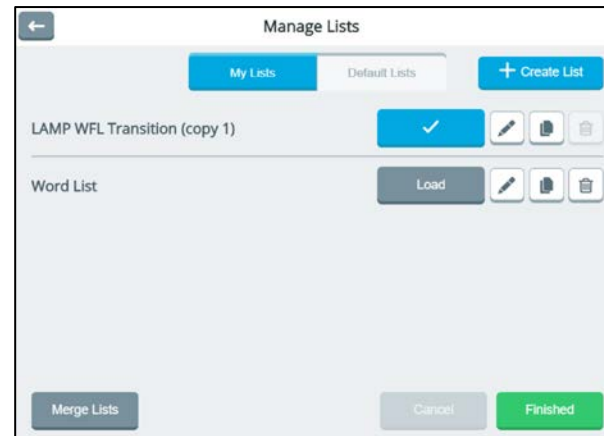

VOCABULARY BACK UP

1. Insert a USB flash drive into a USB port.
2. Select **Menu**, and then **Toolbox**.
3. Select the **System Settings** tab.
4. Select **Export & Import**.
5. Select **Export Selected Files**.
6. Select the **Vocabularies** tab.
7. **Export** your desired vocabulary.
8. Navigate to your USB flash drive.
9. Rename the file. It is a good idea to include the date using xx-xx-xx. For example, LAMP WFL FULL Backup 3-5-19.

CONNECT TO Wi-Fi®

1. Select **Menu**, and then **Toolbox**.
2. Select the **System Settings** tab.
3. Select **Devices & Connections**.
4. Select **Choose** and then connect to a network.
5. To return to Empower, select the PRC logo  that appears in the taskbar at the top.

UPDATE SOFTWARE VIA Wi-Fi

1. Select **Menu**, and then **Toolbox**.
2. Select the **System Settings** tab.
3. Select **Updates**.
4. Select **Check for Updates** next to Update Using Wi-Fi and follow the prompts.

RESTRICTIONS

You can lock/unlock the menu two ways.

To lock/unlock via a password:

1. Select **Menu**, and then **Toolbox**.
2. From the **User Settings** tab, select **Restrictions**.
3. Select **Password**. Follow the prompts to create a password.

To lock/unlock via a USB flash drive:

1. Select **Menu**, and then **Toolbox**.
2. From the **User Settings** tab, select **Restrictions**.
3. Select **USB**. Insert a USB flash drive and follow the prompts. This process places an unlock key on the USB flash drive. Anytime this flash drive is plugged into the device, the Menu button will be unlocked.
4. **Remove the USB flash drive**, and the Menu button will be locked.

Learn: Register for a **LAMP Workshop** at...

<https://www.aacandautism.com>

Teach: Find a vast library of **therapy materials** at...

www.AACLlanguageLab.com

Accelerate: Monitor **progress** and **language growth**...

www.realizelanguage.com